BAT KT7 (USA) BAT KT8 (International)

Charger for PockeTalker 2.0

QUICK SETUP GUIDE

BAT KT7

(Power Supply & Cable)

BAT KT8 (BAT KT7 + 3 Adapters)

Power Supply Safety Warnings

1. Never charge the unit in enclosed spaces such as handbags, cases, or other closed storage bags.
 2. Unplug this device during lightning storms, or when charging is complete and the device is unused for long periods of time.
 3. Use correct adapter for country of use.
 4. Use only the power supply provided by the manufacturer of this device. Other power supplies may have similar specifications, but may not be equivalent. Use of an unapproved power supply may leave the device partially or completely inoperable, and will void the warranty.
 5. Protect the power cord from being walked on or pinched, particularly at plugs, receptacles, and near the power jack on the unit.
-

Product Description

The BAT KT7 and BAT KT8 are designed for charging the Williams Sound PockeTalker 2.0 when used with NiMH batteries.

The **BAT KT7** is the USA version with no adapters.

The **BAT KT8** is the International version with supplied adapters for the UK, Europe, and Australia.

WARNING: USE THIS CHARGER WITH WILLIAMS SOUND NiMH RECHARGEABLE BATTERIES ONLY.


DO NOT USE non-rechargeable batteries, such as alkaline and carbon zinc, **WHICH** may heat up and burst, causing possible injury and damage to the equipment.

BEFORE CONNECTING THE CHARGER TO THE POCKETALKER, MAKE SURE THE POCKETALKER HAS NiMH RECHARGEABLE BATTERIES INSTALLED.

Setup and Operating Instructions

1. Make sure you have the correct adapter for the country of use.
 2. Open the battery compartment and insert two AAA NiMH batteries. Be sure to follow correct polarity.
 3. Set the battery switch (located inside the battery compartment) to the NiMH position.
 4. Plug the USB A Plug into the USB A Port on the power supply.
 5. Plug the USB Micro-B connector into the charging port on the PockeTalker (Figure 1).
 6. Plug the power supply (with correct international adapter, where necessary) into an AC wall outlet. An indicator light on the power supply will illuminate to indicate the power supply has AC power.
 7. The charging indicator on the PockeTalker will blink slowly while the batteries are being charged, then stay on when the batteries are fully charged.
 8. It will take approximately 16 hours to fully charge the batteries.
-

Figure 1: PockeTalker 2.0 Charging Connection

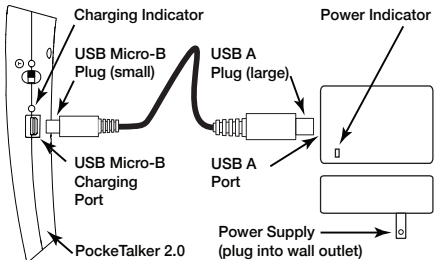Warranty

The BAT KT7 and BAT KT8 are covered by a 90 day warranty.

Customer Service

If you experience difficulty with your system, call Toll-Free for Customer Assistance:

1-800-328-6190 (U.S.A.) or

+1 952 943 2252 (Outside the U.S.A.)

If it is necessary to return the system for service, your Customer Service Representative will give you a Return Authorization Number (RA) and shipping instructions.

10300 Valley View Rd • Eden Prairie, MN 55344
800-328-6190 / 952-943-2252 • FAX: 952-943-2174
www.williamssound.com

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©2015 Williams Sound LLC