

# Infrared Products Bulletin

## Using Infrared Products with Plasma Monitors

### TECHNICAL BULLETIN

This technical bulletin is to inform Williams Sound customers that some Plasma monitors may interfere with Infrared (IR) systems.

The interference appears as a buzzing noise heard in the IR receiver when it is placed close to the plasma monitor. Increasing the IR signal strength can help reduce the Plasma interference. This can be done by adding IR emitters to a system and/or increasing the distance between the IR receiver and Plasma monitor.

Also, please note when using the small area TV Infrared Listening system, using the microphone input on the IR system with Plasma monitor speakers can cause noise in the IR system.

If you have any questions regarding this information, please contact Williams Sound at:

**1-800-843-3544**